# Silber 70

## Solo

DMG Chemisch-Pharmazeutische Fabrik GmbH  
 Elbgaustraße 248 · 22547 Hamburg · www.dmg-dental.com  
 091835/#1\_2013-03

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Deutsch

#### Produktbeschreibung

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### English

#### Product description

Non-gamma-II-amalgam Silber 70-Solo is a pliable restorative material for stress bearing restorations in posterior teeth. Silber 70-Solo is offered in self-activating, predosed Surlyn® pillows for use in a reusable capsule.

#### Indication

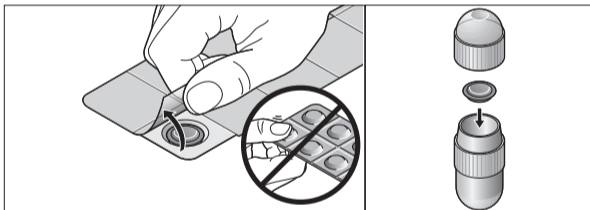
Amalgam should only be used in stress bearing restoration in posterior teeth (Class I and II), if other restorative materials or restoration techniques are not indicated.

#### Basic safety instructions

- For dental use only!
- Keep out of reach of children!

#### Recommended use

#### Preparation of Silber 70-Solo

Silber 70-Solo is supplied in self-activating, predosed Surlyn® pillows, which can be individually removed from the blister package.

To do this, peel the cover cardboard of one application starting from the edge and discard. Remove the amalgam pillow, put it into the reusable two-piece capsule (the classical, plug-in type) and close the capsule again.

Never try to press the amalgam from the packaging. When opening the package, please be careful not to damage the plastic packaging of the applications.

#### Trituration

- ▶ Clamp the capsule into place in the mixing unit and mix according to directions. The capsules are automatically activated during mixing; manual activation is not necessary. Mixing time and speed considerably influence the consistency of the amalgam. Mixing units recommended for trituration/mixing include:

Silamat (Ivoclar/Vivadent)	approx. 8 s
Vari-Mix (Caulk) [M2]	approx. 8 s

Correctly trituated amalgam will form a silvery, pliable, cohesive mass without any further processing.

#### Removal of the mixed amalgam

- ▶ Simply pull the two parts of the reusable mixing capsule of Silber 70-Solo apart and remove the amalgam.
- ▶ If necessary, gently tap the capsule to dislodge any material that is trapped within the capsule due to settling.

#### Placement

1. Dry the cavity. Protect the areas close to the pump with a liner. The use of rubber dam is recommended.
2. Placement of the material occurs immediately after completion of mixing. While still in a pliable condition, small portions of Silber 70-Solo are placed into the cavity and condensed. Only apply enough pressure as is necessary to achieve a good adaptation. The condensation in approximal cavity areas requires the use of an appropriate matrix system.
3. The cavity should be initially somewhat overfilled. Then remove the surplus mercury-rich material, shape the restoration and check occlusion with articulating paper (e.g. blue paper, occlusion foil). After approx. 2 hours the restoration has fully regained its capacity to withstand stress.

#### Times

Condensation time	≤ 4:30 min
Carving time	≤ 5:30 min

#### Polishing

- ▶ As a rule, final polishing with a suitable polisher for the amalgam is done after 24 hours at the earliest. Permanent and sufficient cooling with water spray is necessary.

### Care of the reusable capsule

- ▶ The reusable capsule may be gently cleaned with a cloth and a little alcohol. Please ensure that the inside of the capsule is completely dried. For safety reasons, do not use the capsule more than 50 times (packaging unit).

### Important guidelines for handling amalgam

- Avoid skin and eye contact with the material. Wear suitable mouth and eye protection and gloves. If amalgam gets into contact with the eyes, rinse them thoroughly with water and if necessary contact a doctor.
- Do not ingest the material. If ingested, seek advice from a doctor.
- If moisture penetrates the amalgam before it has cured, properties such as strength and corrosion resistance could be adversely affected. If the alloy contains zinc, this contamination may lead to an excessive expansion (delayed expansion). If possible, always apply in a dry environment.
- Restoring the cavity without a suitable pulp/dentin protection may irritate the pulp. This applies especially to deep or medium deep cavities, which have not been lined.
- Pressing surplus amalgam into the sulcus gingivae during condensation of the restorative material leads to damage of the tooth retaining tissues in this area.
- According to the present level of knowledge, there is no proof for risks of embryo-fetal toxicity caused from presence/placement and/or removal of amalgam restorations. It is, however, recommended not to have an extensive amalgam therapy (placement and especially removal of amalgam restorations) during pregnancy.
- In individual cases, local reactions of the mucosa have been observed. These could be irritations or allergic reactions. In case of allergy to components of dental amalgam, the use of suitable alternative materials must be considered.
- Unnecessary exposure to mercury vapor during handling, placement or removal of dental amalgam should be avoided. Placement and removal of dental amalgam should be performed with appropriate water spray and vacuum suction.
- Mercury can be injurious to health if it is not handled correctly. Mercury vapors are toxic when inhaled and this effect is cumulative. Immediately clean up mercury spills, including places that are hard to reach. Use a plastic syringe for sucking up spills. Smaller spills may first be covered with sulfur powder and then removed. Small isolated drops can be collected with tin foil (Sn) and removed. Inhalation of the vapor must be avoided. During and after cleaning the area in which the mercury was spilled must be thoroughly ventilated.
- **The waste material and all primary containers which contained mercury must be disposed taking suitable precautions. Used capsules are to be disposed of according to national regulations.**
- Special metals and their alloys are corroded by mercury and become brittle. Unnecessary contact between mercury and these metals is to be avoided.

### Contraindications

Do not use on children, pregnant women or patients with severe renal deficiency or known allergies to any of the ingredients. Amalgam is unsuitable for retrograde or endodontic restorations, as a core build-up material and as a restorative material for cast crowns.

### Side effects

In individual cases, hypersensitive reactions (allergies) and/or electrochemically caused local reactions (e.g. taste sensations and irritation of the mucosa) have been observed. After placement or removal of dental amalgam restorations, increased mercury concentration in blood or urine has been observed. According to available scientific knowledge, this increase has not been associated with any adverse health effects.

### Interactions with other materials

If there is contact (approximal or antagonistic) to crowns, bridges, inlays or restorations of other alloys, galvanic effects can occur. In most cases, they will be of short lasting duration. If the effect persists, the user should consider replacement of the dental amalgam restoration with another material.

### Composition

The alloy powder contains 69.3% silver, 19.4% tin, 10.9% copper and 0.4% zinc. It consists of splinter-shaped and spherical particles.

	Size	Alloy powder	Mercury
Silber 70-Solo	1	380 mg	435 mg
Silber 70-Solo	2	560 mg	640 mg

### Storage

- Store in a dry place and protect against moisture. Do not store over 25 °C!
- Do not use after the expiration date.

### Classification

Silber 70-Solo fulfills the requirements of EN ISO 13897:2004.  
EN ISO 24234:2004 + A1:2011.

### Packaging

<b>Silber 70-Solo, Capsule size: l = 33.0 mm ø = 13.5 mm, 50 Applications, 1 Mixing Capsule</b>	
Size 1	REF 509581
Size 2	REF 509582

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