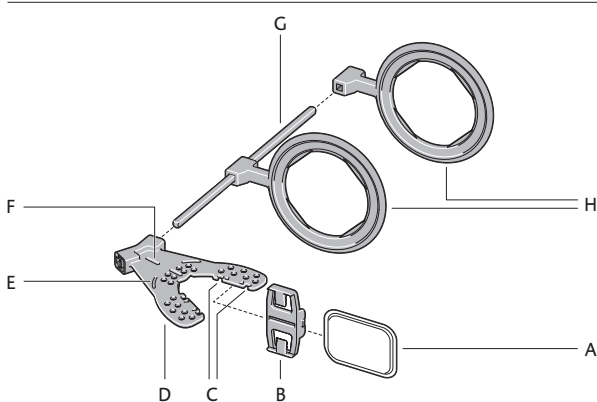
Icon X-ray Holder

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English

Product description

The X-ray film holder system Icon X-ray Holder is used for positioning the head of a patient relative to the X-ray device being used as well as for reproducing the image position. This enables X-ray images in a reproducible position over a longer period of observation. Assessment along with follow-up checks in the case of proximal lesions are thus greatly facilitated. The customization of the bite plate allows repositioning of the plate. The image is taken in the same position. Using the scale on the X-ray film holder, the image quality of the X-ray image can be assessed. The scale is used for baseline assessment of the extent of the lesion and the comparative assessment of the image (superimposition of X-ray images).

Intended purpose

Customizable bite plate for bitewing X-rays of adults and children.

Limitations of use

X-rays can be harmful to the health of pregnant patients! If the patient is pregnant or might be pregnant, avoid X-rays that are not urgently needed.

Indications

Loss of tooth substance due to caries.

Contraindications

Do not use the aid where there are known allergies to any of the components or contact allergies.

Patient target group

People treated in the course of a dental procedure.

Intended users

Dentist, dental assistant staff

Notes for use

- If the material is applied into the patient's mouth with a single use applicator, the single use applicator must only be used on one patient for hygiene reasons.
- The dental impression material remains on the bite plate for later follow-up checks.
- The bite plate is allocated to one patient for the entire duration of the treatment.
- The X-ray film holder is autoclavable and can be used for several bite plates.
- For selection of the size of bite plate, please refer to "Recommended Use," steps 5 and 6. Bite plate size 1 should be used for adults in most cases. Bite plate size 2 should be used for patients with jaws that are particularly narrow.

Recommended use (for image, see »Handling«)

A X-ray film (not included in the pack)	E Curved line
B X-ray film holder with scale (0.1 – 0.8 mm)	F Center marker
C Catches	G Guide rod
D Bite plate	H Sighting ring

1. Position the patient sitting upright in the X-ray unit and make sure that the patient's back is up against the backrest.
2. Put on a lead apron plus thyroid shield if necessary.
3. Clip the X-ray film holder [B] onto the desired side of the bite plate [D], moving it in the direction of the arrows until it is in the catches [C].
4. Slide the X-ray film [A] into the X-ray film holder [B].

Note: Be sure that the X-ray film is slid up to the end stop in the pre-fabricated groove in the bite plate.

5. Insert the chosen size of bite plate with the X-ray film holder fitted and X-ray film in place into the patient's mouth.
6. Alignment and size selection of the bite plate:

Note: Align the bite plate so that the curved line [E] shown on it follows the curve of the front teeth.

Note: Align the bite plate so that the center marker [F] on the bite plate is centered between the two incisors.

7. Remove the bite plate with the fitted X-ray film holder plus film.
8. Using a silicone adhesive spray (e.g. DMG Tray Adhesive), moisten the areas of the bite plate where individual blobs of the dental impression material are to be applied.
9. For adults (see note below for children), apply a small area of dental impression material on the upper and lower side of the bite plate in the region of the first molar and incisors.

Note: Make sure that the catches for the X-ray film holder remain free of the dental impression material.

Note: Follow the instructions below for applying dental impression material in children.

Applying the dental impression material to primary dentition:

- Apply blobs in the area of the deciduous molars and incisors.

Applying the dental impression material to mixed dentition:

- Apply blobs in the area of the first permanent molars and deciduous canines.

Note: Make sure that the catches for the X-ray film holder remain free of the dental impression material.

1. Insert the bite plate (now with dental impression material) into the patient's mouth and ask the patient to bite down on it.
2. Once the material has set, remove the bite plate. Any excess dental impression material can be removed with a scalpel.

Note: Make sure that the catches for the X-ray film holder remain free of the dental impression material.

3. Insert the guide rod [G] into the corresponding hole in the bite plate in the direction of the arrow up to the end stop and check that the guide rod is firmly in place.

- Grind the sighting rings [H] to fit the X-ray tube.
- Insert the sighting rings in the desired position on the guide rod.
- Apply the X-ray holder system by securing the sighting rings on the X-ray tube.
- Reposition the bite plate in the patient's mouth. Start taking the X-ray image.

Cleaning and Sterilization:

X-ray film holder, guide rod, sighting rings

- Disassemble the X-ray film holder system.
- Remove any residues from the parts using hot soapy water.
- Perform steam sterilization.

Note: There is a risk of the materials being damaged as a result of improper sterilization.

- Autoclave removable plastic and metal parts in separate sterilization bags.
- Autoclave parts at a maximum of 134°C (273.2°F).
- Follow the instructions of the autoclave's manufacturer precisely!

Bite plate and dental impression material

- Disassemble the X-ray film holder system.
 - The bite plates and dental impression materials should be disinfected by immersion in a commercial disinfectant solution.
- DMG recommends the use of a 2% glutaraldehyde solution.
 - Follow the instructions of the disinfectant solution's manufacturer precisely!

Interactions

No interactions with other materials are currently known.

Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components of the material cannot be ruled out.

Warnings / precautions

- Keep out of the reach of children!
- Reuse: Standard surface disinfection as required. No specific maintenance required.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Performance characteristics of the device

Holding device for X-ray films

Storage / disposal

- Disposal according to national regulations.
- No special storage conditions required.

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