LuxaPrint Ortho Flex

DMG Digital Enterprises SE
 Elbgaustraße 248 · 22547 Hamburg · Germany · www.dmg-dental.com
 092926/2022-02

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Produktbeschreibung

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English

Product description

Light-curing resin based on (meth)acrylate for the production of dental devices. For DLP / SLA printers that work at wavelengths of 385 nm. Instruments with material parameters already defined in their systems are recommended: e.g. DMG 3Demax/DMG 3Delite (DMG), D10+/D20+/D30+/D50+ (RapidShape), P10+/P20/P30+/P50+ (Straumann).

Intended purpose

Manufacturing of bleaching trays and transfer trays (in case of indirect bonding of brackets) using 3D printing.

Limitations of use

Only insert LuxaPrint Ortho Flex intraorally in a fully polymerized state.

Indications

- Bleaching trays: Tooth discoloration
- Transfer splints: Indirect bonding of brackets in orthodontic treatment.

Contraindications

Do not use the material where there are known allergies to any of the components or contact allergies.

Patient target group

Persons receiving treatment as part of a dental procedure.

Intended users

Dentist, dental technician

Notes for use

- During handling (e.g. transferring), the material should be exposed to daylight / ambient light only for as short a time as possible, since lengthy exposure to light can lead to unwanted curing of the material.
- Wear nitrile gloves, protective clothing and goggles when working with the material/printed object.

For optimum properties of the end product:

- Ensure that the instruments used during the production process are in proper condition and comply with machine and material parameters.
- Ensure clean working procedures. Residues on the instruments can cause defects on the printed object.
- Use the material parameters already defined in the system for the cleaning units DMG 3Dewash (DMG), RS wash (Rapid Shape) and P wash (Straumann).
- Allow objects to cure completely.
- Use the material parameters already defined in the system for the light-curing units DMG 3Decure (DMG), RS cure (Rapid Shape) and P cure (Straumann).
- Thinner layers increase the accuracy of the printed object and increase printing time.
- Position the printed object as flat as possible on the printer's build platform. A flatter positioning increases accuracy and reduces printing time.
- Maximum curing depth during post-curing: For solid objects and when light-curing on both sides, the material can be up to 7 mm thick (at a curing depth of 3.5 mm).

Recommended use

Note: Observe the device and material parameters.

Note: Shake the bottle for at least 1 min before use.

Note: Maintain a processing temperature of 23 °C ± 2 °C (73.4 °F ± 3.6 °F).

Note: For optimal process reliability, scan the RFID code of the material (supported by DMG 3Demax/DMG 3Delite (DMG), D10+/D20+/D30+/D50+ (RapidShape), P10+/P20+/P30+/P50+ (Straumann)).

Finishing the printed object

Production steps	
Recommended drip-off time after raising the platform: 10 minutes.	
Note: Finishing should take place immediately after the construction process where possible.	
1.	Carefully remove the printed object from the build platform. When using the printers DMG 3Delite (DMG), D10+ (RapidShape) and P10+ (Straumann), start directly from step 2.
2.	Cleaning: Clean the printed object with isopropanol (approx. 99 %) using an appropriate cleaning instrument and specified program (e.g. DMG 3Dewash, RS wash or P wash).
3.	Post-curing: Post-cure the printed object using the appropriate light-curing unit and the specified program (e.g. DMG 3Decure, RS cure or P cure).
4.	After removing the support, finish the surface.

Interactions

Residues of methacrylate-based materials (uncured material residues) may influence the setting behavior of silicone impression and bite registration materials.

Residual risks / side effects

There are no known side effects to date. The residual risk of hypersensitivity to components of the material cannot be ruled out.

Warnings / precautions

- Keep out of the reach of children!
- Avoid skin contact with the liquid material and components prior to post-curing! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Avoid breathing in the fumes.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Observe the safety data sheet.
- Do not use the material where there are known allergies to any of the components or contact allergies.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Performance characteristics of the device

Cured material	
Shore-D hardness	≤ 80

Uncured material	
Viscosity	≤ 4.0 [Pa*s]
Light sensitivity to fluorescent lamps	≤ 5:00 [min]

Storage / disposal

- Store in a dry place at room temperature (15 – 25 °C/59 – 77 °F) and protected from light!
- Even low exposure to light can trigger polymerization.
- Always keep the container tightly sealed; carefully seal immediately after every use.
- Do not use after the expiration date!
- The cured material can be discarded together with household waste following consultation with the disposal company and in compliance with regulations.
- Incinerate the resin mixture as hazardous waste in accordance with official regulations.

Composition

Aliphatic urethane acrylate, IBOMA, TPO, additives. Contains TPO.

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